

ARISTA™ COVID-19 Antigen Rapid Test (Nasal Test)

93.3% SENSITIVITY **100% SPECIFICITY**



Nasal specimen



Fast results in 15 mins



High accuracy



Simple to use



MOH endorsed

QUICKER AND MORE CONVENIENT TESTING THAN EVER BEFORE.

The ARISTA™ COVID-19 Antigen Rapid Test is a screening tool for self-testing for early Covid-19 detection. Accurate, fast and simple to use with a less invasive sample collection method and get your result within 15 minutes. Able to detect 31 mutated variants including Alpha, Beta, Gamma, Delta etc. Approved by the **Medical Device Authority (MDA)**.

HIGHLY SCALABLE FOR SAFER SCHOOLS, WORKPLACES, AND OTHER SPACES.

Paired with the **Viva Life App**, you have the option to securely store all your records on your phone and generate a digital certificate whenever required. It's all in your hands.



VERIFIABLE DIGITAL TEST CERTIFICATES

Viva Life Platform

ARISTA Biotech have developed an enterprise solution to facilitate the re-opening of businesses with safer spaces for all.



Individuals can generate and store ready-to-display verifiable digital test certificates via the Viva Life App



Organisations can oversee the test status of their people via the Viva Life Enterprise Account



TEST KIT CONTENTS



Sterile swab (1x)



Tube with diluent (1x)



Dropper with protective cap (1x)



Plastic Biohazard Disposal Bag



Instructions for Use (1x)




Test cassette (1x)


HOW TO USE

1 – Anterior nasal sample collection.

- Remove the sterile swab from the peel-pack being careful to avoid touching the sponge tip with your hands or contacting it with any foreign surface.
- Insert the swab sponge tip 1.5cm (3/4 inch) into your nostril.
- Slowly rotate the swab in a circular path against the inside of your nostril at least 4 times for a total of 15 seconds.
- Using the same swab, repeat steps (B) and (C) in the other nostril.

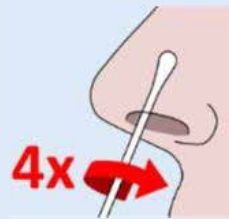

Left nostril - 15 seconds

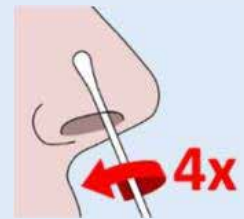

Right nostril - 15 seconds

2 – Preparing the sample specimen.

- Fully immerse the swab sponge tip into the diluent. To ensure all the sample from the swab is mixed into the inserting diluent, rotate the swab stick in a twisting motion in the tube diluent for 30 seconds while squeezing tube against the swab sponge tip.



- Continue to squeeze the sides of the tube while removing the swab so as to extract all the liquid from the swab sponge tip. Then safely dispose of the swab.



- Hold the dropper by the protective cap while long-pointed end into the tube until it is securely fitted.



3 – Conducting the test.

- Gently mix the sample again by 15-30 rotating the tube for 5 seconds stirring motion. Then remove the protective cap from the dropper.
- Remove the test cassette from its foil pack (if you haven't previously done so).



- Invert the tube and squeeze 3 drops into the smaller rectangular sample well at the bottom of the test cassette (not the larger rectangular C/T results window at the top of the cassette). **Note: you may continue the test even if you inadvertently applied 4 drops.**




- Start the stopwatch and read the test results after 15 minutes.

Note: after 30 minutes, test results no longer considered valid.



4 – Interpreting test result.


Negative test result

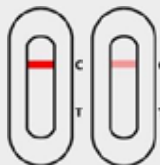

If the "T" line is blank and the "C" line displays a visible colour band (even if faint) then the result is negative and you are unlikely to have COVID-19.

Note: Negative results are presumptive should not be used as the sole basis for infection control decisions, particularly in the presence of clinical signs and symptoms consistent with COVID-19, or in those who have been in close contact with the virus. In these cases, it is recommended that the test is repeated after 1-2 days. In all cases, a definitive diagnosis must be made by a doctor and local health and safety control measures should always be adhered to.

Positive test result

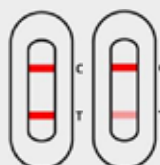

If both the "T" line and "C" line display visible colour bands (even if faint), then the test result is positive and you most likely are infected with COVID-19.

Note: Positive results require strict adherence to your local public health authority's guidelines for suspected infections - including confirmatory PCR screening at your local hospital or testing station, self-isolation, and reporting to the applicable authority always be adhered to.

Invalid test result



If the "C" line is blank, then the test result is considered invalid.

Note: Invalid results are represented by a complete absence of a visible "C" line. In this case, carefully read the "Instructions for Use" and repeat the test. If a repeated test result is invalid, then contact your local product supplier.

WWW.VIVALIFE.APP

VIVA LIFE
Verifiable
digital test
certificates





Viva Life Platform

ARISTA Biotech have developed an enterprise solution to facilitate the re-opening of businesses with safer spaces for all.



Individuals can generate and store ready-to-display verifiable digital test certificates via the Viva Life App



Organizations can oversee the test status of their people via the Viva Life Enterprise Account

CLICK TO DISCOVER

1 * Viva Life is compatible with the full range of products from ARISTA Biotech



Viva Life App – Apple App and Google Play Store

The screenshot shows the registration form for the Viva Life App. The form is titled 'Register' and includes the following fields:

- First Name*: Sarah
- Last Name*: Summer
- Email Address*: test40@iceangelid.com
- Password*: (empty)
- Organisation ID: 111101 (highlighted with a red box)
- Employee ID: (empty)

Below the fields, there is a checkbox that is checked, with the text: "By registering for Viva Life, I acknowledge the [PRIVACY POLICY](#) and accept the [TERMS OF USE](#)".


At the bottom of the form, there is a purple button labeled 'REGISTER' and a link: "Already have an account? [SIGN IN](#)".

Organization ID

When added, the user's records will automatically be synced to the specific organization's dashboard.

Step-1

Add certificate

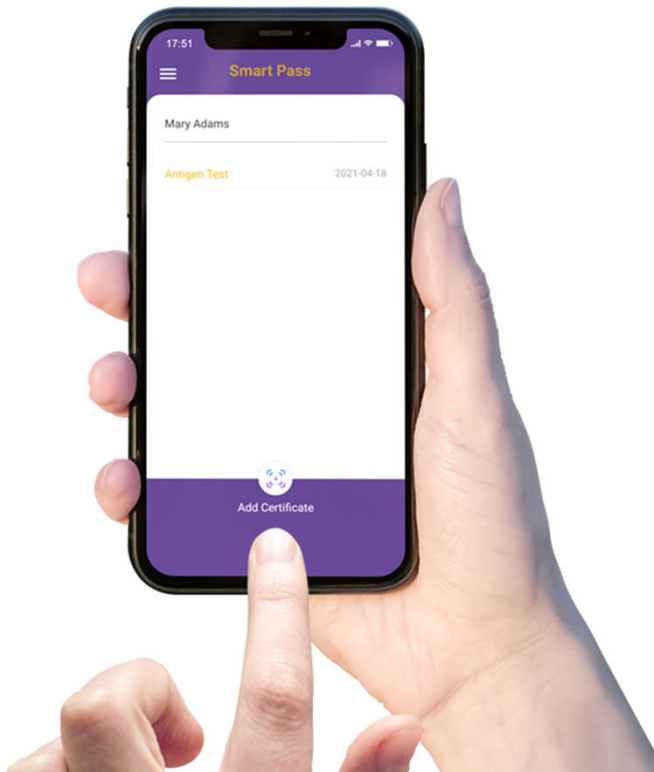


Step-2

Scan QR code




Step-3

Upload photo of result

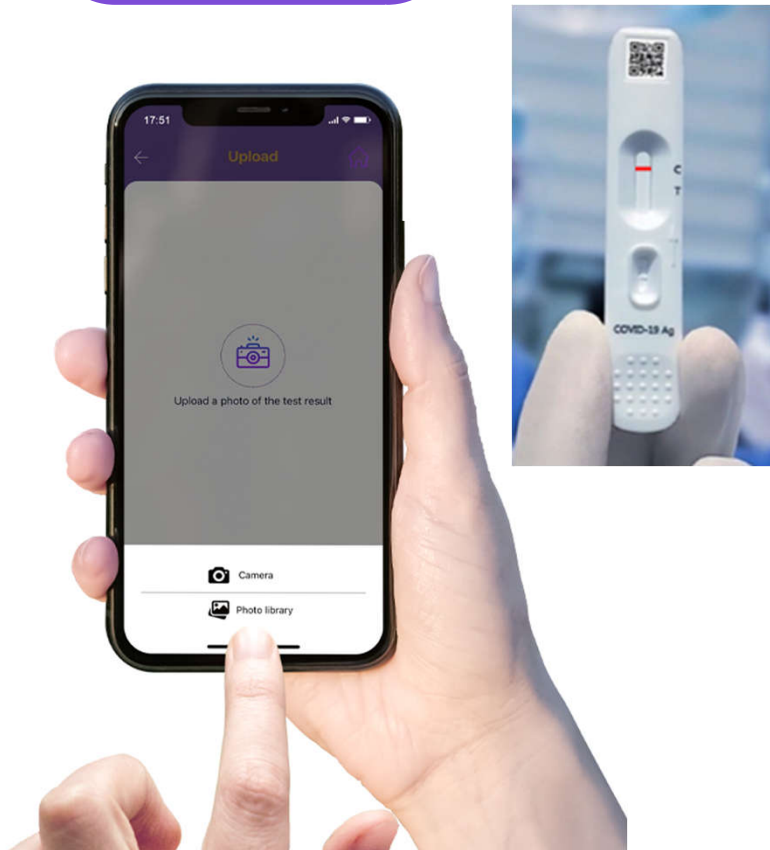


Step-4

Confirm date & result

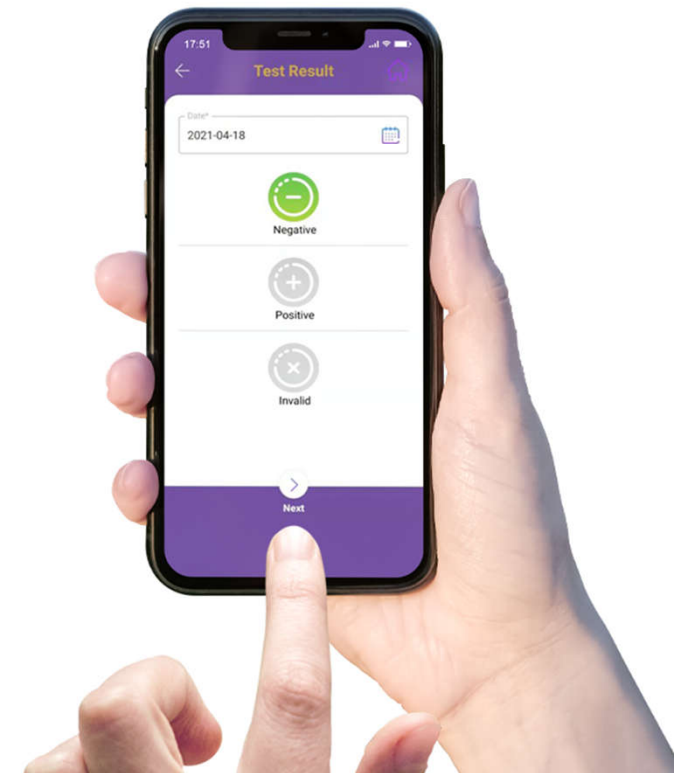

Step 5

Preview & save record

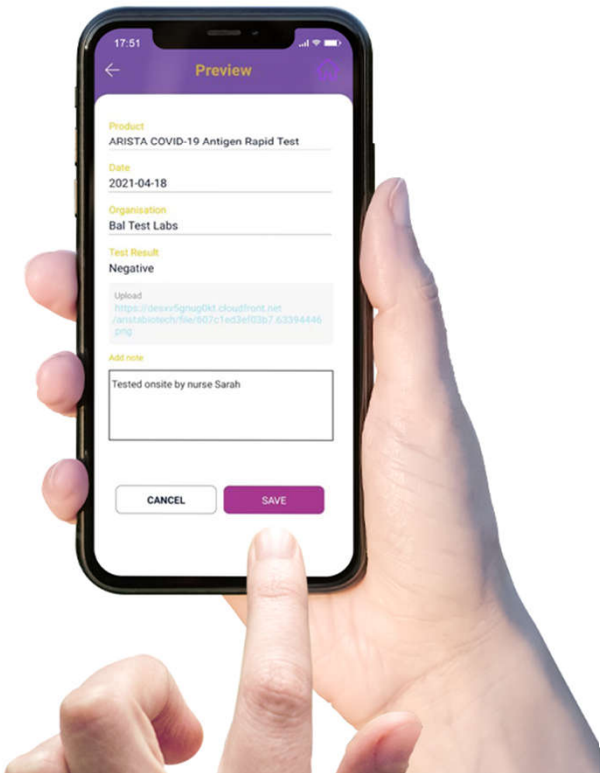

Smart Pass

Mary Adams

Antigen Test	2021-08-14
Nucleic Acid Test	2021-04-21
Antibody Test	2021-02-18

Add Certificate


Success

View digital certificate



Records

View comprehensive test history and delete records

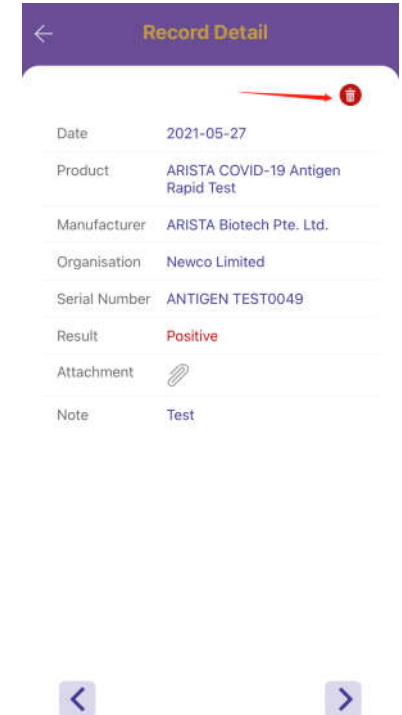




Viva Life App offers individual users an easy to access Smart Pass to simplify their lives by creating digital test certificates for convenient display.

Viva Life Enterprise Account

Viva Life Enterprise Account is compatible with the Viva Life App and syncs individual users' records to an organization dashboard, enabling the organization to facilitate safer environments for their people.

The enterprise account can integrate one or more organization dashboards where each dashboard can represent a separate branch or department of the enterprise – with login privileges to permit access to different levels of enquiry.

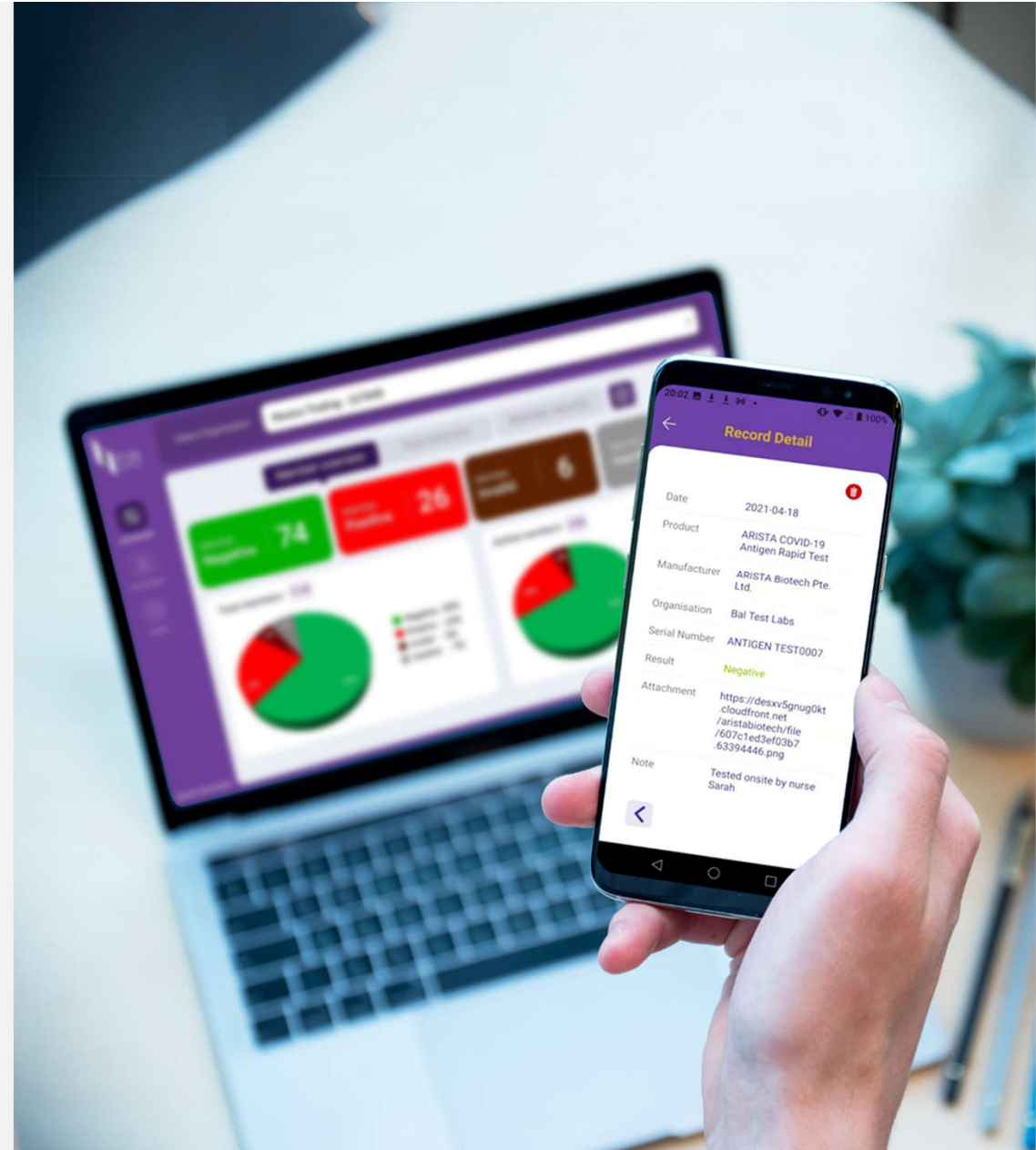


Organization Dashboard



Member Overview


A summary of total members linked to the organization dashboard categorized by:

- i. Active members: tested in the past 7 days categorized by Negative / Positive / Invalid results
- ii. Inactive members: tested more than 7 days ago

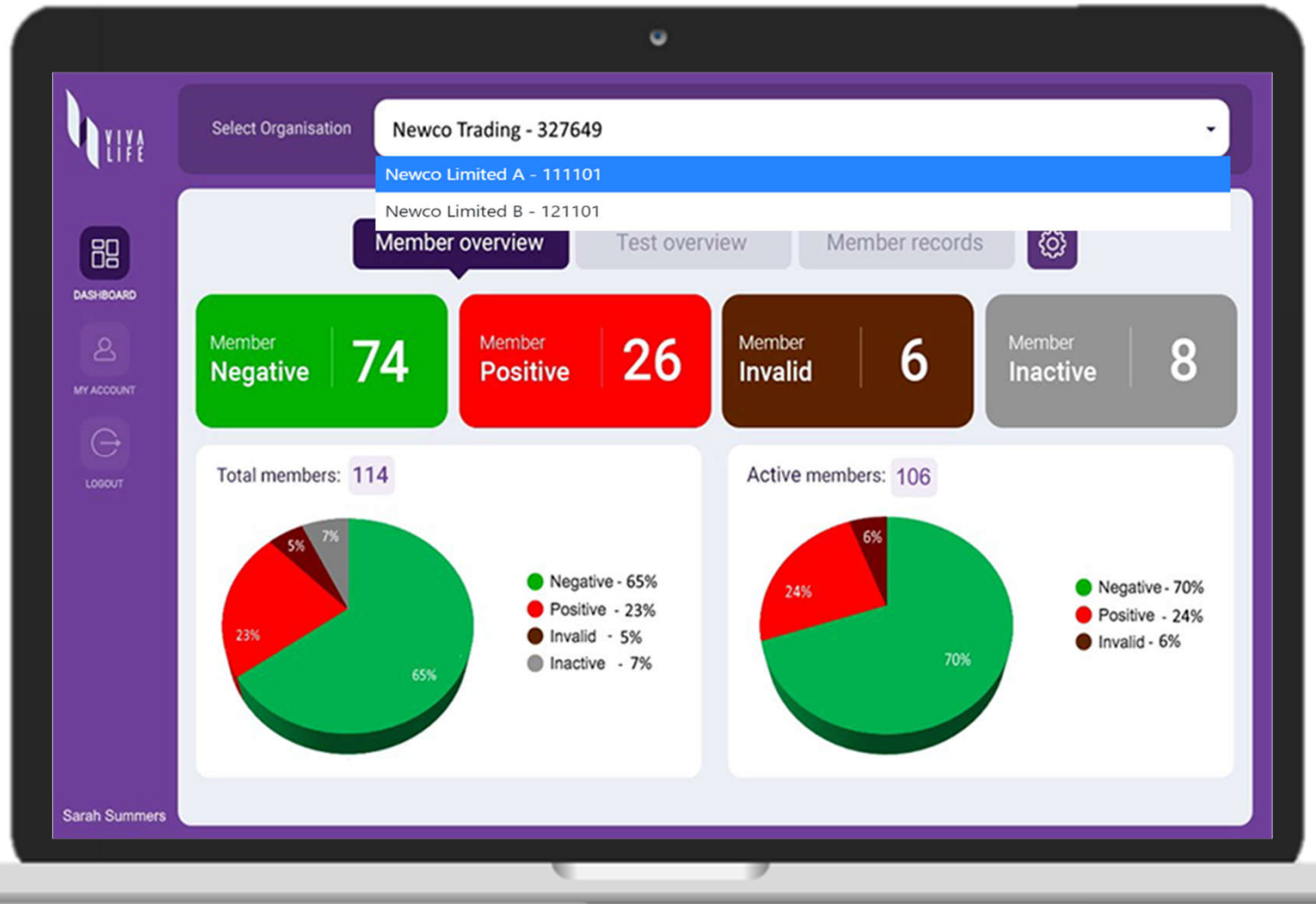


Test Overview

A summary of total tests linked to the organization dashboard categorized by Negative / Positive / Invalid results within a user-selected date range.

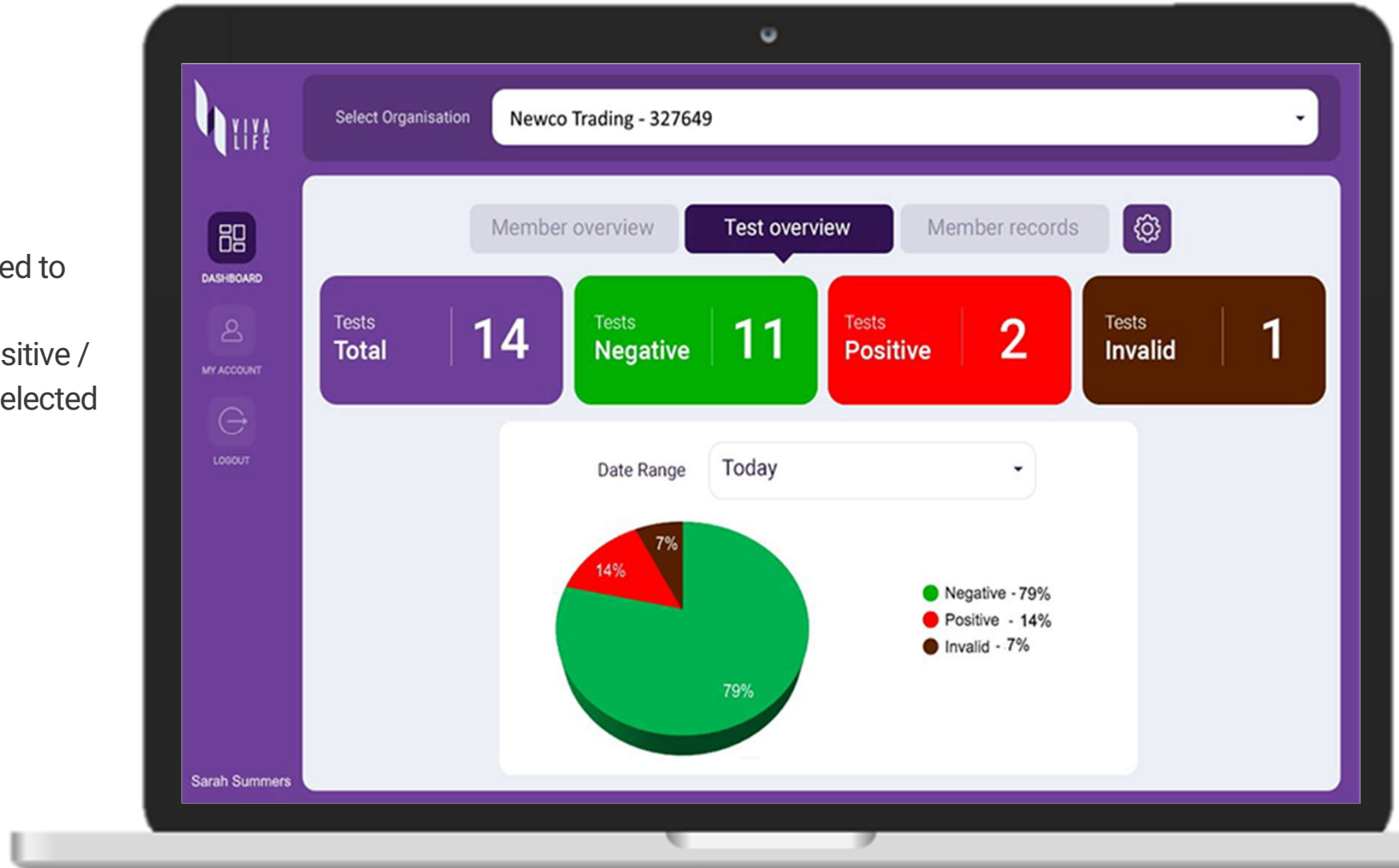


Member records

Detailed drill-down records of all members linked to the organization dashboard.

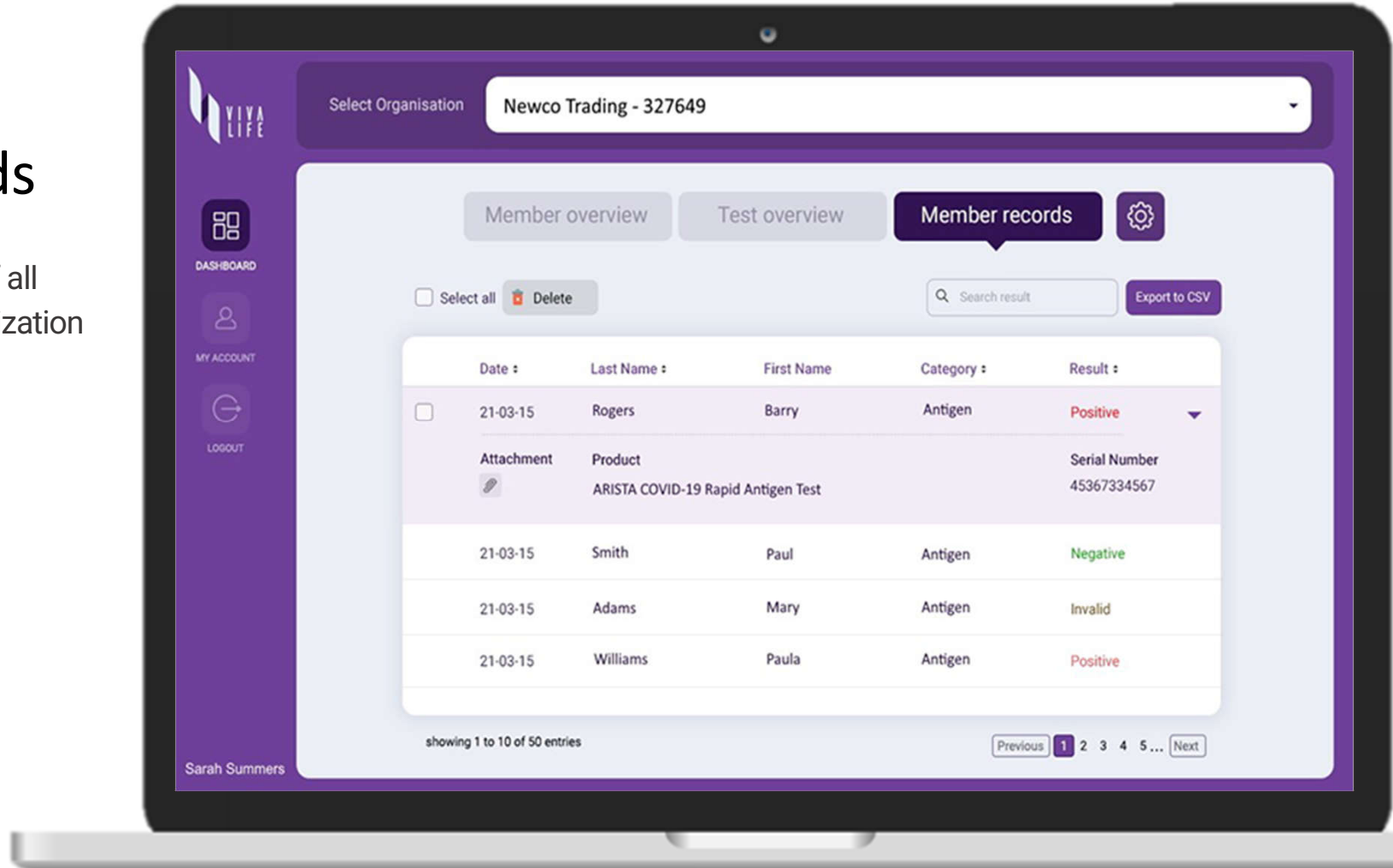


Dashboard Settings

Inviting additional Admin or Overview users to the dashboard - or removing user access to the dashboard

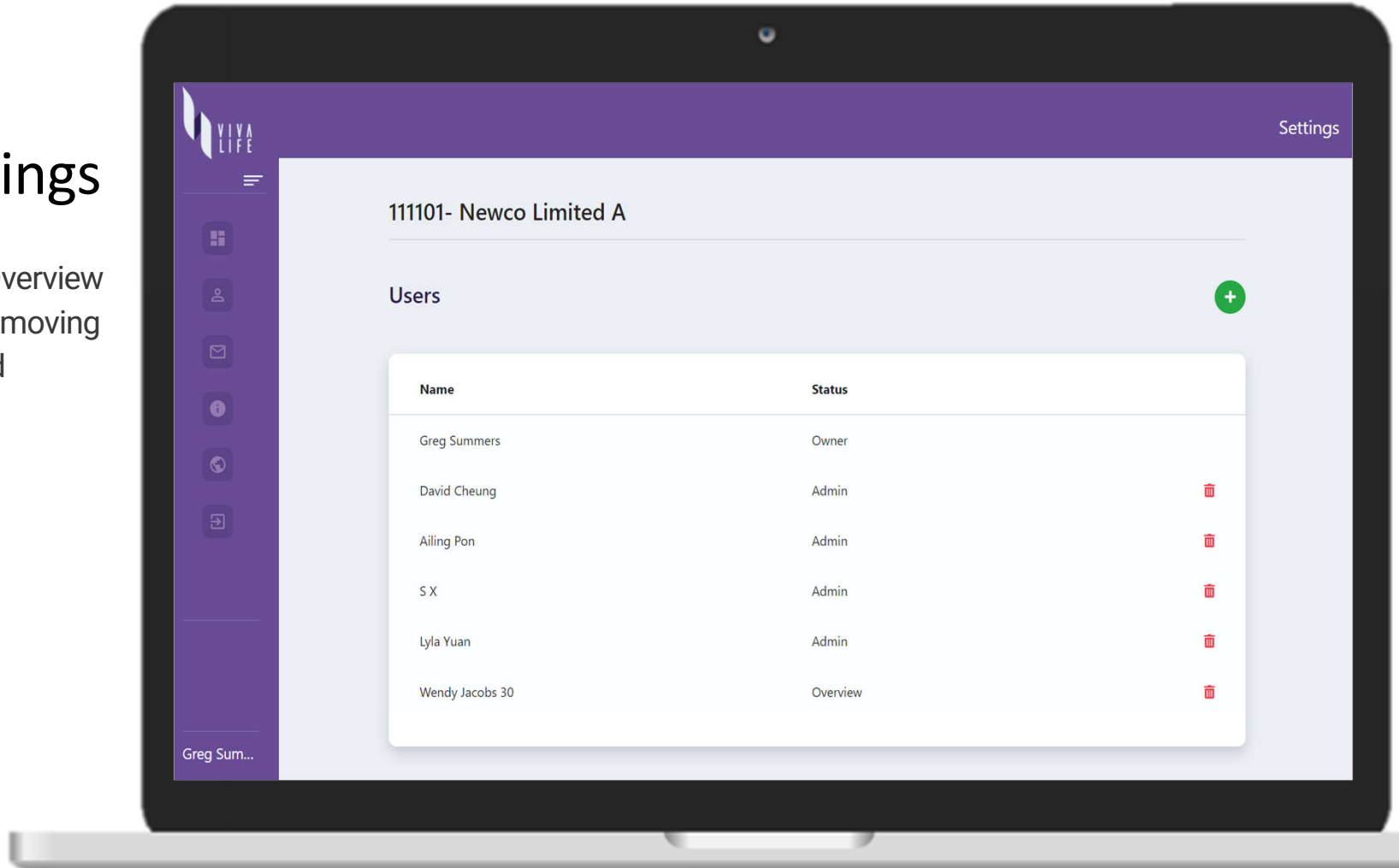

Message Center

Invitation notifications and status updates

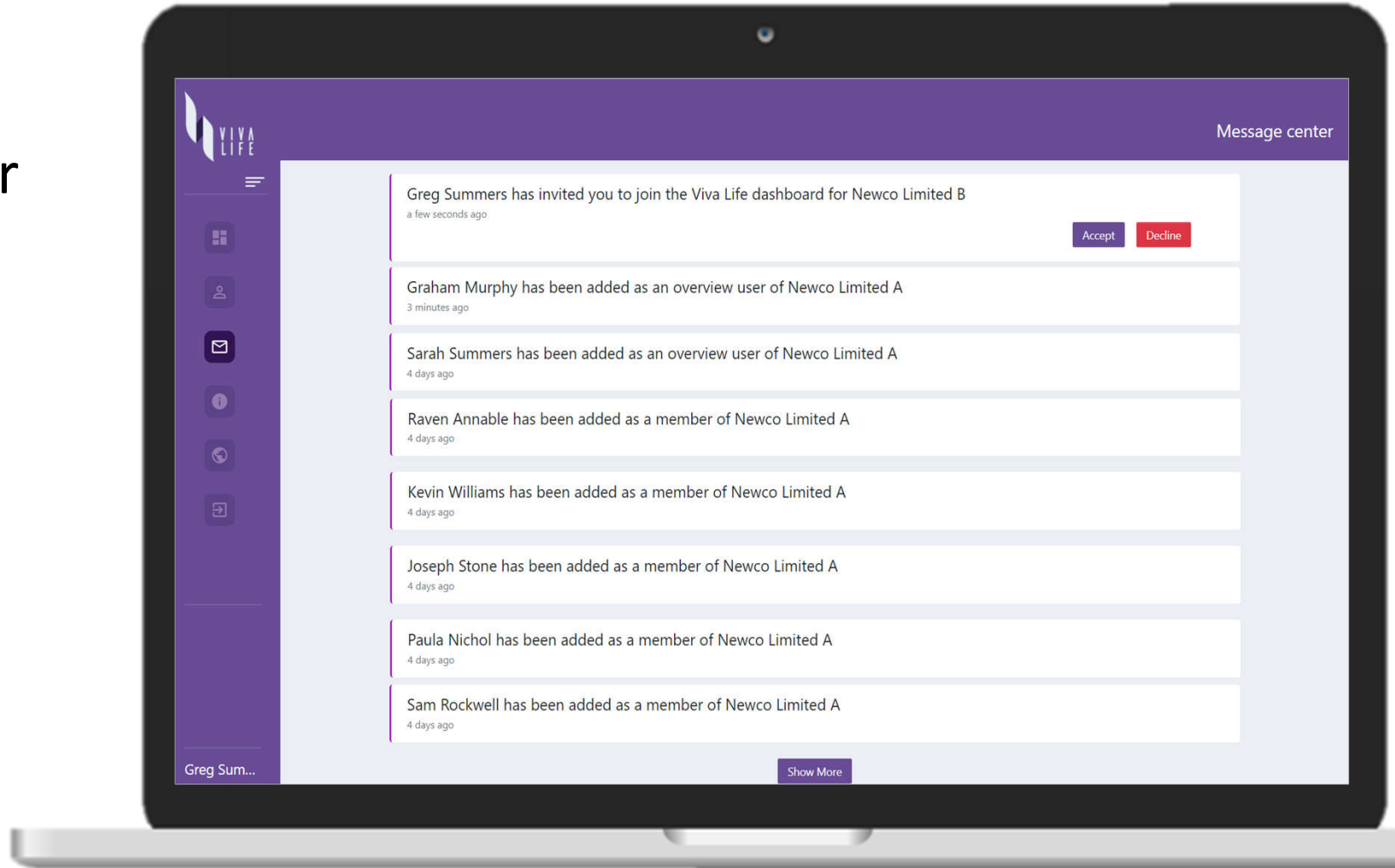



The Viva Life Enterprise Account facilitates the management of the organization's surveillance testing program and provides instant data feedback to enable immediate steps to limit the risk of business shutdown.